


Smart Medication & Vitamin
Wi-Fi Pill Dispenser
With UV Blocking Lid



Instruction Manual
#70678

1. Contents	2
2. Safety Tips & Precautions	3
3. Device Overview	4-7
4. Device Setup	8-12
5. Operating Device	13-14
6. Connecting to App	15-18
7. Warranty	19
8. Care & Maintenance	20
9. Specifications	21
10. FCC Statements	22-23

What's in the box

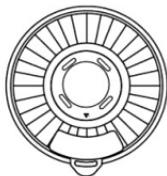

Smart Medication & Vitamin
Wi-Fi Pill Dispenser



Locking Key



Dosage Rings (x9)



Power Adapter



Power Cable



AA Alkaline
Batteries (x4)



User Guide

Safety Tips & Precautions

This instruction manual is meant to ensure the safe operation of the product. Non-compliant use may void the warranty.

- Keep devices away from children.
- Use only manufacturer-supplied and recommended attachments.
- Keep the device away from open flames and heat. Fire, electrical shock, damage, or injury may occur.
- Keep the pill dispenser on a flat, stable surface. Dropping may cause damage to the dispenser or pills. Flip it only if you're removing pills. The Internal pill container is removable for easy refilling.
- Water can damage the dispenser, batteries, or the medication inside. Do not immerse or expose this pill dispenser to water or other liquids. Keep in a cool, dry environment.
- Do not fill the internal pill container unless it is completely dry.
- The user, caregiver, or healthcare provider is solely responsible for organizing medications and assigning quantities and times.
- MOBI is not liable for missed medications.
- If a dose is missed, contact your pharmacist or healthcare provider for instructions.
- Some medications should be kept in an airtight container. Ask your healthcare provider or pharmacist if you're unsure if this dispenser is right for you.
- Change the batteries immediately when a low battery symbol  appears in the upper right corner of the LCD.
- Never use mismatched, old, or damaged batteries.
- Never force the locking cover open.
- Do not mix Alkaline, Standard, or Rechargeable Batteries.

Device Overview



Understanding Daily Dosage Rings

Your smart pill dispenser comes with 9 Daily Dosage Rings corresponding to how frequently medication needs to be taken daily.

The rings help properly dispense medication from the internal pill container and include 28 daily medication dosage slots.




First, place the **dosage ring** in the center of the **internal pill container** with the tab facing the bottom (indicated by the ▼ icon on the plastic container.)

Then align the **two triangle marks** on the dosage ring and faceplate, then fold the tabs to secure the dosage ring before filling the dispenser with medication.

Once the dosage ring is correctly situated, the **alternating colors** help distinguish between different days. For example, if you use the **2x per day** ring (meaning medication only needs to be dispensed twice daily), the internal pill container will hold 14 days' worth of medication.

The grey spaces to the right of the orange tab represent 1 day's worth of medication. The following white spaces represent the upcoming 2nd day's medication. The dosage rings will alternate between grey and white spaces to distinguish between different days. Each colored space includes numbered slots representing the different alarm times configured during the device setup process.

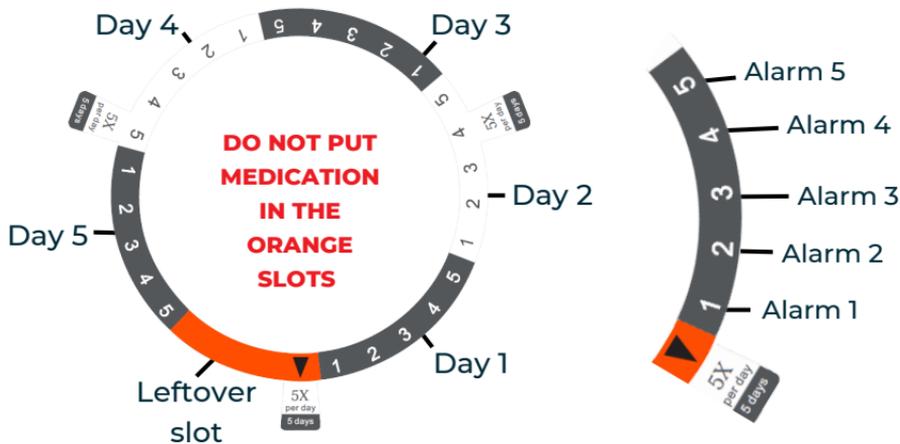

If you need medication dispensed 5 times during the day, the ring can accommodate 5 days or 25 doses, with three leftover slots. Leftover slots are indicated by the orange spaces on the ring and are substantially larger orange areas on the 5x per day and 6x per day rings.

Reminder: **Do not put medication in leftover slots.**

Choose the Daily Dosage Ring with the tab that shows how often you need medication dispensed daily. Position the ring around the compartment. Fill each compartment accordingly, then place the carousel back into the dispenser with the dosage ring.

Ring	# of Meds per Day	Programable Table Period
1	1	28 Days
2	2	14 Days
3	3	9 Days
4	4	7 Days
5	5	5 Days
6	6	4 Days
7	7	4 Days
8	8	3 Days
9	9	3 Days

Plugging in the Cable

1. Connect the power cable to the pill dispenser.
2. Plug the other end into a USB port.

NOTE: The cable should always be plugged in, as it is the primary power source for this device.

Installing the Batteries

1. Turn the dispenser upside-down. Insert the locking key into the battery cover and turn clockwise.
2. Remove the battery door.
3. Insert four (4) AA alkaline batteries inside the compartment.
4. Replace the battery cover. Insert the key into the cover and turn counterclockwise to lock.


NOTE: Replace batteries if the low battery symbol appears.

Batteries are installed as backup power in case of power outages. The device will stay on for approximately (4) days when powered by batteries.

Locking and Unlocking the Cover


The cover lock helps prevent accidental spills and unauthorized access, and **can also help with confused patients who may try to take medication incorrectly.**

1. Align the locking key with the lock hole (as shown.)
2. To lock; turn the key clockwise while pressing down on the cover latch.
3. To unlock; Turn the key counterclockwise.
4. Lift the cover open using the cover latch.

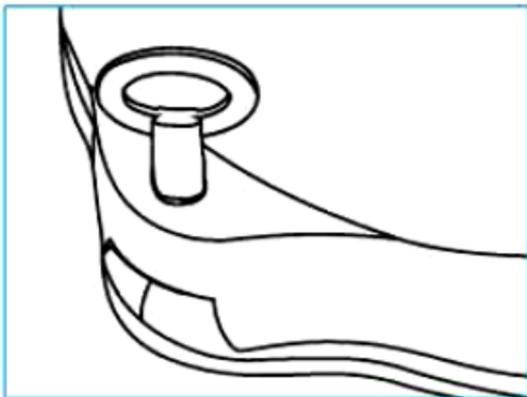


Selecting 12-Hour/24-Hour Mode

Press and hold the CLOCK button for (2) seconds to switch between 12 and 24-hour mode.

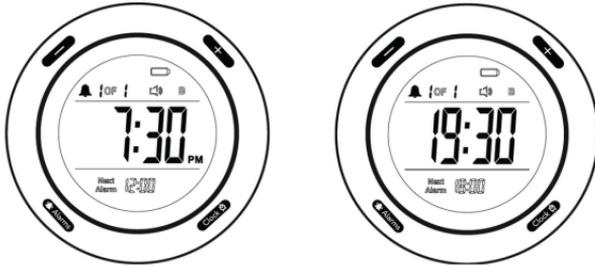

Setting the Time

1. Press the CLOCK button once to start the time setting.
2. Press the + or - buttons to adjust the hour. Then, press the CLOCK button again to adjust the minute.

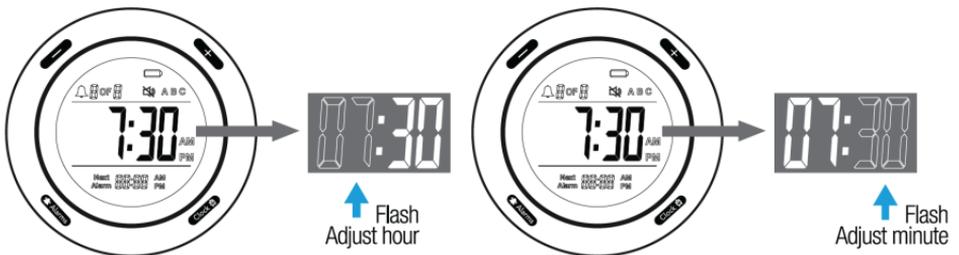

3. Use the + or - button to adjust the minute, similar to the hour.


NOTE: Pressing and holding the + or - button will cause the hour or minute digits to move faster.

4. Press the **CLOCK** button to exit time settings with changes saved, or press and hold the **CLOCK** button for two (2) seconds to cancel the setting change and exit out of the time setting.

NOTE: The digits on display will flash to indicate which digits are currently being changed.

Setting the Alarm

1. Press the "ALARM" button to start the alarm time setup.
2. Press the "+" or "-" to select between an alarm, then press the "ALARM" button to confirm.
3. Press the "+" or "-" to adjust the hour. Then, press the "ALARM" button to confirm the hour and set the minute.
4. Press the "+" or "-" button to adjust the minute, similar to the hour.

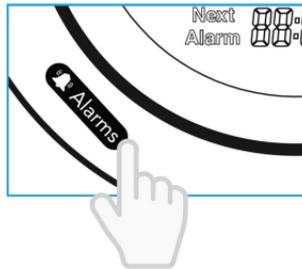

NOTE: Pressing and holding the "+" or "-" button will cause the hour minute digits to move faster. The digits on display will flash to indicate they are currently being changed.

5. Press the "ALARM" button once or press and hold down the "ALARM" button for two (2) seconds to save and exit alarm setting mode.

To cancel setting change, wait a few seconds for alarm setting mode to end automatically.

To delete current alarm settings, press and hold the "CLOCK" button while in alarm setting mode until  is shown on the display screen.

Setting the Alarm Tone & Volume


1. Press and hold the "ALARM" button for three (3) seconds to select alarm tones. On-screen alarm tones are displayed as A, B, and C.
2. Press and hold the "-" button for three (3) seconds to change volume levels.

NOTE: Volume level is represented by On-Screen "speaker" symbol.

The three (3) alarm tones have distinct beeping patterns:


- **Alarm A:** 20 seconds of continuous beeping, followed by 10 seconds of silence, then 20 seconds of continuous beeping as the pattern repeats.
- **Alarm B:** Three sets of three beeps. The beeps will stop for 7 seconds, and then resume beeping, as the pattern repeats.
- **Alarm C:** One (1) beep every 3 seconds.

NOTE: All alarms will continue their patterned beeping for 30 minutes before stopping. Also, all alarms will sound with the same alarm, until it is manually changed to a different tone.

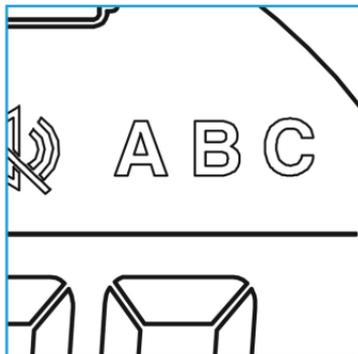

Reset Factory Settings

Press and hold the "CLOCK" and "ALARM" buttons simultaneously for five (5) seconds to clear and reset all settings.

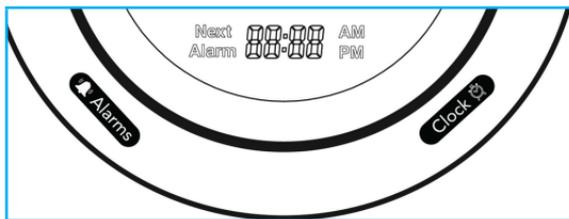

Care and Maintenance

- Wipe the exterior with a soft, damp cloth. Do not use harsh chemicals.
- If needed, remove the internal pill container and gently rinse/wipe the individual pill compartments.
- Allow the container to dry fully before replacing or refilling it.

Operating the Smart Medication & Vitamin Wi-Fi Pill Dispenser

1. Choose the Daily Dosage Ring that corresponds to your needs (*please refer to "Understanding the Daily Dosage Rings" on page 5*).
2. Situate the Daily Dosage Ring in the center of the internal pill container and place your medications in the container to correspond with when they are taken (*1st, 2nd, 3rd, etc.*) indicated by the numbers on the ring (*1, 2, 3, etc.*) **Do not place any medication in the orange sections.** When the orange is approaching the dosage window, it is time to refill the container.
3. Install the batteries (*refer to "Installing the Batteries" on page 8*).
4. Unlock the cover using the key (*please refer to "Locking and Unlocking the Cover" on page 8*) and place the internal pill container inside the main body.
5. Set the time (*please refer to "Setting the Time" on page 9*).
6. Adjust the alarm volume and alarm tones, and choose a 12-hour or 24-hour time (*please refer to "Setting the Alarm Tone and Volume" on page 11*) and (*"Selecting the 12-Hour/24-Hour Mode" on page 9.*)

7. Set your medication times (*refer to “Setting the Alarm” on page 10*).
8. Close the cover and lock it with the provided key (*please refer to “Locking and Unlocking the Cover” on page 8*).
9. At designated medication times, the alert will sound.
10. Slide open the window by pushing the tab to the right.
11. Tilt the pill dispenser to pour the pills into your hand or medication cup. Make sure all pills have come out of the compartment. The alarm should stop automatically.
12. Close the dispensing window and place the pill dispenser on a flat surface.

Connecting to App

- 1.Ensure you have access to your email to verify your MOBI Smart Account.
- 2.Have your 2.4 GHz Wi-Fi Network Name & Password ready.


If you experience issues setting up your device, contact support at www.getmobi.com/gethelp.

You can also reach us at (323)904-3000, Monday - Friday, 8:30 AM - 5:30 PM PST

Please Note: This product requires a 2.4 GHz Wi-Fi signal. Signal strength may vary based on location, bandwidth, and network strength.

Download the MOBI Plus App



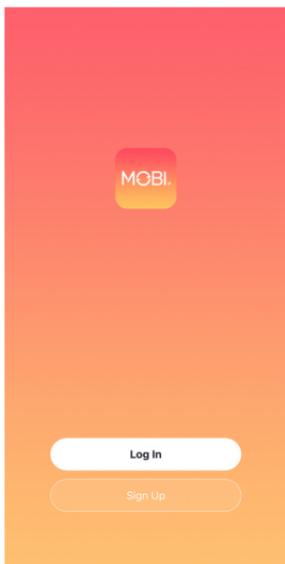

Register


If you are a first time MOBI Smart user, please create an account.


1. Open the MOBI Smart App & tap "Register."
2. Agree to the Privacy Policy terms & conditions.
3. Enter a valid email address & verify.
4. Create a password and login

Add Device

Once you've accessed your MOBI Smart account move to the main Device tab.

- Tap "Add Device" or tap on the "+" icon on the top right hand corner.
- Tap on "Health Monitoring"
- Select the Smart Pill & Medication Dispenser

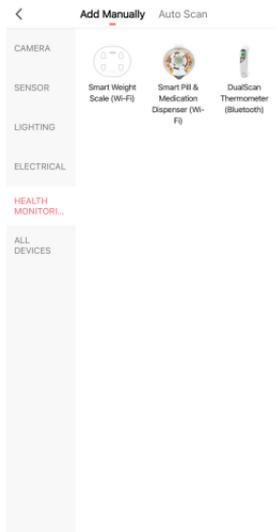

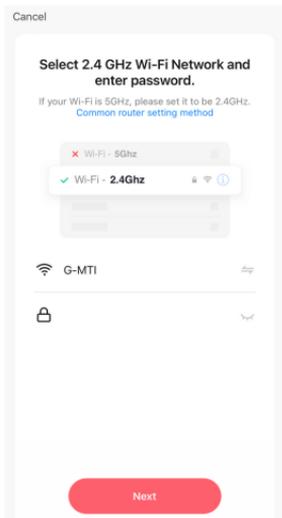

Setup 2.4 GHz Wi-Fi

- Enter your 2.4 GHz Wi-Fi name & password.
- Tap "Next" to confirm.


If you cannot identify your 2.4 GHz Wi-Fi Network, please contact your Internet Service Provider.

After confirming, you'll be prompted to reset your device.

Connect to Hotspot

Once you've reset your device and tapped "Next," you'll need to connect your mobile device to the Smart Pill Medication Dispenser Hotspot.

- Tap "Connect" to be redirected to your Wi-Fi settings.
- Once on your Wi-Fi settings, tap on the available signal appearing as: "SL-MOBI WiFi-XXXX."
- After connecting, return to the MOBI Smart App to continue.

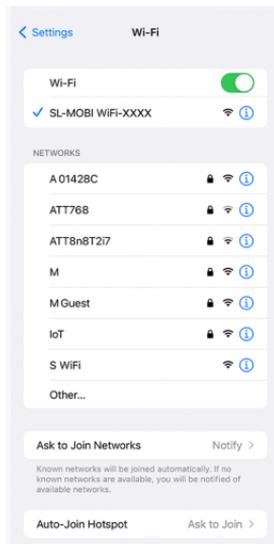

Done

Added.



Smart Pill Medication Dispenser

Device added successfully

Living Room

Master Bedroom

Second Bedroom

Dining Room

Kitchen

Study Room

Success

If your device has successfully paired, you'll hear a voice confirmation from the Smart Pill Medication Dispenser.

Once connected, you may rename your device for easier management, or edit at a later time.

Tap "Done" to be taken to the Hub controls.

Scan this QR code to register your product on our website for a FREE warranty extension.



If you experience any issues, please contact us at www.getmobi.com/gethelp.

You can also reach us at (323)904-3000 between Monday through Friday, 8:30 AM - 5:30 PM PST.



Limited Product Warranty

MOBI® (or "THE COMPANY") warrants its products to be free from defects in material and workmanship for 90 days from the date of purchase. Should the product prove to be defective during the warranty period, we would either replace or repair the product at our option without charge. After the warranty period, a service charge will apply for replacing parts and labor costs. To obtain warranty service, you must return the product complete with its original parts & accessories and a copy of the original purchase, gift, or registry to us. This warranty does not cover damage caused by tampering, accident, misuse, abuse, improper maintenance, unauthorized modification, or connection to an improper power supply. There will be a charge for repair caused for such damage. This warranty excludes all incidental or consequential damages and any liability other than stated above. MOBI reserves the right to modify or discontinue the offer by posting a notice on the app or website.

Care and Maintenance

1. Wipe the exterior with a soft, damp cloth. Do not use harsh detergents.
2. If needed, remove the internal pill container and gently rinse/wipe the individual pill compartments. Allow the container to dry fully before replacing it or refilling it.
3. Do not immerse the outer pill dispenser in water or any other liquid.
4. Make sure that the internal pill container is clean and dry before refilling it.
5. Please follow local and state guidelines for proper battery disposal.

Specifications

1. Dispenser length: 9.25"
2. Dispenser width: 9.25"
3. Dispenser height: 3"
4. Packaging length: 9.3"
5. Packaging width: 9.6"
6. Packaging height: 3"
7. Weight: 1.9lbs.
8. Power: 4x AA alkaline batteries (included)
9. Product type: Pill dispenser
10. Approved location: Indoor use only
11. Adult assembly required: No
12. . Additional tools required: None
13. Number of compartments: 28
14. Number of dosage rings: 9
15. Connectivity: 2.4 GHz Wi-Fi
16. Alarms: Audible and visual
17. Cover lock: Yes
18. Country of origin: China
19. Warranty: Yes

FCC Statement

Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) This device must accept any interference received, including interference that may cause undesired operation.

FCC Radiation Exposure Statement:

This equipment complies with FCC radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with a minimum distance of 20cm between the radiator & your body.

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

This product can be used
with or without
a smart app.



Scan to find the Most Up-To-Date
Instruction Manual for this product.

